SCHEDA TECNICO INFORMATIVA *TECHNICAL SHEET*

Articolo: 01096
Article

Aggiornamento 10.02.2016
Updated

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This briefing has been prepared in accordance with the provisions UNI 10913-2001 (personal protective equipment: note). The content of the notice must be regarded as necessary reference for the information to the end user and therefore the proper use of PPE. Therefore advised to carefully read.

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Definition of the product and the manufacturer

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Disposable latex glove, ambidextrous and beaded; external finish of palm and fingers with microgrip, smooth interior. Powder free. Treated with emollients. Glove with a very low protein content. Non sterile. Ivory color.

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D. Min. 21/3/73 e s.m.i.

Tipologie Alimenti - *Food categories*

Acidi <i>Acids</i>	Alcolici 15% <i>Alcohol 15%</i>	Alcolici 50% <i>Alcohol 50%</i>	Grassi e Oleosi <i>Greases and Oils</i>
NO	OK	OK (30min)	NO

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The production is detected in batches in a sequence order-traceable production. The expiry date of the product is 5 years from date of production after which it begins to reduce its tensile properties and resistance.

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6.3 Ean, ITF, unità logistiche
Ean, ITF, Logistic units

	XS	S	M	L	XL
Ean Code	8000697031914	8000697031822	8000697031839	8000697031846	8000697031921
ITF Code	08000697003393	08000697003348	08000697003355	08000697003362	08000697003409
Colli per strato (EPAL) <i>Cartons per layer</i>	9				
Strati per pallet <i>Layers per pallet</i>	7				

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The glove is a non sterile device. It is a category III personal protective equipment, and it has been certified by CIMAC 0465. Check previously for conditions of use which includes submitting and if in doubt contact your dealer reference.

The glove can be used in: industry environment, catering, laboratory, health & medical, beauty treatments & spa, cleaning and professional hygiene & household, leisure. It has biological barrier function. Wear the gloves with dry hands. Hold the glove on the cuff with one hand and slide it down the other one till the wrist and the fingers are properly dressed (for the purpose of maintaining the values of protection and comfort

the size of the glove needs to perfectly match with user hands).

WARNING: The glove contains NATURAL RUBBER and it may cause allergic reactions in susceptible individuals. In case of irritation or disease attributable to this cause immediately consult a doctor.

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Keep the product in a cool and dry place, away from heat sources and direct sun rays. Excessive exposure to sunlight and/or UV resulting from fluorescent bulbs creates deterioration. Gloves impaired do not emit harmful substances.

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After opening the box, close the box with the appropriate window in order to protect the product from dirt, dust or contaminants.

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The product disposal should follow the current regulation in the country of competence.